LP/H Lack LP/H Lack antibac LP/W Lack

LP/H lacquer LP/H lacquer antibac LP/W lacquer

Sicherheitshinweise / Safety regulations:

Für LP/H Lack / For LP/H lacquer:

DE: Enthält: Methyl-methacrylat, Propensäurederivate, Vinylester Resin
EN: Contains: Methyl methacrylate, Acrylic acid derivatives, Vinylester Resin

Für LP/W Lack / For LP/W lacquer:

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General product information

LP/H lacquer and LP/H lacquer antibac:

All known hard acrylic earmold materials can be coated with the egger LP/H lacquer and LP/H lacquer antibac.

Delivery units:	20 ml glass bottle with brush	REF 30603
LP/H lacquer	50 ml glass bottle with brush	REF 30600
	100 ml	REF 30601
	250 ml	REF 30602
	500 ml	REF 30605
	1000 ml	REF 30609
LP/H lacquer antibac	50 ml glass bottle with brush	REF 30621
	100 ml	REF 30622
	500 ml	REF 30624

LP/W lacquer:

All known hard acrylic earmolds can be coated with the LP/W lacquer in order to improve the wearing comfort for the customer.

These light-curing lacquers can be polymerized in the egger UV-curing units EL 1, EL 2, EL 3, and EL 4 as well as in other light-curing devices.

Delivery units:	20 ml glass bottle with brush	REF 30703
	50 ml glass bottle with brush	REF 30700
	100 ml	REF 30701
	250 ml	REF 30702


Notes for storage:

Store the material in a cool and dry place.

Always keep the container closed.

Protect from light.

If stored correctly, the product can be stored and used until: see date of expiry on the product.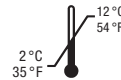

Notes for usage:

- Close the container tightly after use.
- Protect against light during usage.
- Avoid direct skin contact with the liquid material! The hands should be protected by wearing appropriate gloves before working on the earmold and before using the lacquers.
- While working with the product wear protective glasses!
- Ingredients of the LP/H lacquer, LP/H lacquer antibac, LP/W lacquer may prompt allergic reactions in persons with a likely disposition. In such a case the material should no longer be used.

Notes for processing:

The lacquers are to be processed by skilled persons only and in accordance with the regulations of the safety data sheet. Lacquering operations must always be carried out under a suction unit. Waste disposal instructions can be seen in the relevant safety data sheet of the product.

Processing

1. Preparation

Roughen the surface for example with corundum paper (grit 180) REF 40704.

CAUTION: In order to avoid unevenness always roughen the entire surface of the earmold and coat it completely with the lacquer.

Fix the earmold on a piece of nylon tube REF 97100 (for BTE earmolds) or on a piece of Erkogum REF 30911 (for ITE earmolds).

To obtain a dust and greaseless surface, clean it with a surface cleaner REF 31600 ff.

2. Handling

Apply the lacquers by using a varnishing brush or immerse the earmold and then remove any excess lacquer by shaking the earmold.

3. Hardening

One polymerization is sufficient with LP/H lacquers (see table).

If LP/W lacquer and the egger EL 1 plus N₂, which works with inert gas, are used, also only one polymerization is necessary (see table).

If LP/W lacquer without inert gas is used for the polymerization, a second polymerization treatment is necessary:

- after the polymerization carefully wipe off the surface of the earmold with a cloth that is soaked with surface cleaner or
- insert the earmold into the LP-Cleaner bath and repolymerize it in the light-curing unit (see table).

Time of polymerization in the different light-curing units:

Light-curing unit	LP/H lacquer LP/H lacquer antibac	LP/W lacquer Polymerization	LP/W lacquer Repolymerization in the LP-Cleaner
egger EL 1	2–4 minutes	4–5 minutes	4–5 minutes
egger EL 1 plus N ₂	2–4 minutes without inert gas	4–5 minutes with inert gas	–
egger EL 2	3–5 minutes	5–7 minutes	5–7 minutes
egger EL 3	3–4 minutes	3–4 minutes	3–4 minutes
egger EL 4	1–2 minutes	2–3 minutes	2–3 minutes

Rinse the earmold with water and blow it dry after the second polymerization if you used the LP-Cleaner bath.

<p>NOTE: It is also possible to coat ITE earmolds with integrated electronics with LP/W lacquer.</p> <p>Please note that the earmold must not be put into the LP-Cleaner bath. In order to remove the greasy inhibition layer please use a cloth that is soaked with surface cleaner or repolymerize the earmold by using inert gas.</p>
--

Counter indications: Do not use if hypersensitive to acrylate/methacrylate.