Lack signo

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Lacquer signo

Hot-vulcanizing silicone lacquer
for the surface sealing
of silicone earmolds

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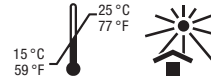
General product information

The lacquer signo is a silicone lacquer for surface smoothing and sealing of silicone based earmolds and impressions, smooth coating – heat curing.

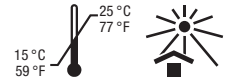
Packaging sizes:

Lacquer signo 250 ml REF 39413

Lacquer signo 500 ml REF 39412

Expiry date and charge:

See bottom of the bottle.

Application

For all silicone based impressions and earmolds. Prior to application (dipping / brushing) the impression must be absolutely clean, dry and free of grease! **Hint:** Use an appropriate surface cleaner (e. g. isopropyl alcohol). After shaking-off the surplus of lacquer, let the impression resp. the earmold (depending on coating thickness and humidity) dry in the oven at 100 °C / 212 °F for approx. 15 min. The surface of the impression resp. earmold shows now a glossy layer.

Caution

Avoid contact with eyes/skin. Wear protective gloves/eye protection.
Do not breathe vapours.

Important

Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of Lacquer signo, do not return residues of lacquer back into original bottle. Do not store in the refrigerator.

Safety advice

On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

PLEASE NOTE: During application of the lacquer use a suction system! For use by trained specialists. Please refer to the safety data sheet!

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EN: Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye damage. Harmful if inhaled. May cause respiratory irritation. May cause drowsiness or dizziness. May cause damage to organs through prolonged or repeated exposure. Toxic to aquatic life with long lasting effects.

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EN: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. In case of fire: Use Carbon dioxide (CO₂), Foam, Extinguishing powder to extinguish. Store in a well-ventilated place. Keep cool.

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**Please follow the product information and
the material safety data sheet.**

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Labortechnik GmbH