# Härterpaste

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### Produktbeschreibung

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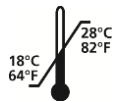
## Directions for use

### Product Description

Hardener paste for condensation vulcanizing impression materials for ear impression taking. Colored for a mixing control when mixing with the impression material.

### Indication for use:

Hardener paste for ear impression material intended to reproduce a patients external auditory canal.

### Processing

Add hardening paste according to the quantity instruction of the respective impression material (approx. 2 resp. 4 %). Mix until an equal colour is obtained.

### Storage

18 –28 °C, Keep tube well closed.

### Item-No.

	Color
	red
5 ml tube	52112
20 ml tube	52212

### Attention

Hardener paste contains organic tin compounds and may irritate the skin. The direct skin contact with the hardener paste has to be avoided as the paste is irritating.

### Hazard statements

- H315 Causes skin irritation.
- H317 May cause an allergic skin reaction.
- H371 May cause damage to organs.
- H373 May cause damage to organs through prolonged or repeated exposure.

### Precautionary statements

- P260 Do not breathe dust/fume/gas/mist/vapours/spray.
  - P280 Wear protective gloves.
  - P362+P364 Take off contaminated clothing and wash it before reuse.
  - P333+P313 If skin irritation or rash occurs: Get medical advice/attention.
  - P405 Store locked up.
  - P501 Dispose of contents/container in accordance with local/regional/national/international regulations.
- Contains: Dioctyltinacetylacetonate, tetrakis (2-butoxyethoxy) silane, dioctyltin oxide, trimethoxyvinylsilane

**Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the material.**

2022-05-10 / Rev. 2  
 Dreve Otoplastik GmbH  
 Max-Planck-Str. 31 • 59423 Unna / Germany  
 Tel.: + 49 2303 8807-0 • www.dreve.com

Warning