## cedis Profi UltraActiv

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For disinfecting and cleaning BTE earmolds and medical instruments for the impression taking – effective against bacteria, fungi, HBV and HIV viruses.

**Exposure time:** 1 minute

**Usage** in a 7.5 % solution (925 ml water + 75 ml cedis UltraActiv): Submerge BTE earmolds or instruments in a 7.5 % solution until completely moistened, let stand for 1 minute, then remove and dry them with a moisture absorbing cloth. Tip: For hygienic reasons, it is recommendable to clean the BTE ear-molds or instruments in the ultrasonic bath before use. For this purpose cleansing glasses designed for ultrasonic devices in different sizes filled with disinfectant have proven useful.

**⚠ Following disinfection, always rub objects dry (e.g. hearing aids, earmolds) that come into contact with skin using an absorbent cloth.**

**Hazard and precautionary statements for the concentrate:**

**H-statements: 314-410** Causes severe skin burns and eye damage. Very toxic to aquatic life with long lasting effects.

**P-statements: 101-102-262-310-280-303+361+353-305+351+338-273**

If medical advice is needed, have product container or label at hand. Keep out of reach of children. Do not get in eyes, on skin, or on clothing. Immediately call a POISON CENTER or doctor/physician. Wear protective gloves/eye protection. **IF ON SKIN (or hair):** Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Avoid release to the environment.

**General safety information:** Use disinfectants safely. Always read the label and product information before use. Dispose of this product and its container at a hazardous or special waste collection point.

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technik GmbH · Aybühlweg 59