Sicherheitshinweise / Safety regulations:

DE: Gefahr
EN: Danger

H: 225 - 304 - 315 - 336 - 411

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EN: Highly flammable liquid and vapor. May be fatal if swallowed and enters airways. Causes skin irritation. May cause drowsiness or dizziness. Toxic to aquatic life with long lasting effects.

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EN: Keep out of reach of children. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing dust / fume / gas / mist / vapours / spray. Use only outdoors or in a well-ventilated area. Avoid release to the environment. Wear protective gloves / protective clothing / eye protection / face protection. IF SWALLOWED: Immediately call a POISON CENTER / doctor. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. IF INHALED: Remove person to fresh air and keep comfortable for breathing. Do NOT induce vomiting. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents / container according to local and national requirements/regulations.

Contains: Hydrocarbons, C7, n-alkanes, isoalkanes, cyclics and n-heptane

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Please follow the product information and
the material safety data sheet.

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General product information

The primer/2 is used for gluing sound tubes into silicone earmolds. Permanent bonding is obtained by pretreating the components with the primer/2 and by gluing them with unigluue/2 (REF 39503).

Notes for storage

Keep container tightly closed in a well-ventilated place. Keep container dry. Store only in the original container. Protect from heat and direct sunlight. Store container upright. The shelf life in unopened original containers is 12 months from the date of dispatch by egger. Shelf life requires storage at temperatures below +23 °C. Keep away from ignition sources. Do not smoke!

Notes for usage

The primer/2 must be processed only by skilled persons and in accordance with the regulations of the safety data sheet. Please see safety data sheet of this product for disposal.

Processing

1. Preparation

Remove dust and harder particles by using compressed air. It is advisable to clean the surface beforehand with Surface Cleaner (REF 31600 ff) from greasy and oily remainders. For the generation of compressed air we offer various compressors and our cedis AirPower spray.

2. Gluing

Apply the primer/2 with a brush onto the sound tube and sound canal bore of the silicone earmold. Then pull the sound tube through the sound canal bore and glue it with unigluue/2 instant adhesive. To achieve the best result, let the bonding dry for approx. 5 to 10 minutes.

Tip: For the application of the unigluue/2 instant adhesive we offer dosing tips in the sizes 0.5 mm (REF 39505) and 1.0 mm (REF 39504).

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Labortechnik GmbH