egger LP/H

klar, rosa - transparent,
beige, flesh, rot, blau

transparent, pink-transparent,
beige, flesh, red, blue

egger LP/HS

klar, rosa - transparent

transparent, pink-transparent

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EN: Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. If on Skin: Wash with plenty of water. If skin irritation or rash occurs: Get medical advice/attention. Dispose of contents/container according to local and national requirements/legislations.

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General product information

LP/H material can be processed in one of the following egger EL devices:

egger EL 1	REF 51100	egger EL 2	REF 51200
egger EL 1 plus N ₂	REF 51110	eLED.LP3	REF 51350

Always observe the operating instructions of the device you are using and the polymerization times indicated below.

Delivery units: See catalog

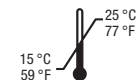
Expiry date and charge: See bottom of the bottle or tube crimp.

Storage: The material should be stored in a cool and dry place, away from foodstuff. Keep container always closed. Protect from light.

For LP/H beige, flesh, red, blue; LP/HS transparent, pink-transparent

For LP/H transparent, pink-transparent

Notes

- Close the container after use.
- In case of air bubbles in the material heat up the open container (but covered with the lid) to a temperature of approx. 50 °C (122 °F) for 15 to 30 minutes.
- Avoid direct skin contact with the liquid material! Hands should be protected with appropriate gloves before working with the material and finishing the blanks.
- The products LP/H transparent and pink-transparent may crystalize (thicken) if stored coolly. The partly crystalized product can be returned to its original state (easily pourable and homogenous) by briefly heating (24 h) it to 40 °C (104 °F). The quality of the material will not be diminished by doing so.

Notes for processing: Before working with the material, it must have a temperature of 23 ± 2 °C (73.4 ± 35.6 °F) at a relative air humidity of 50 ± 5 %.

Processing

Manufacturing of hard BTE earmolds or ITE shells

- Fill the LP/H or LP/HS material into the negative mold made of duplicating material (REF 40900ff). Make sure that the negative mold is completely filled and bubble free.
- Cover it with black foil (REF 53200 or REF 53100).
- Put the filled negative mold into the UV-polymerization unit and let it polymerize according to the following polymerization times. Inert gas is **not** required if you work with the UV-polymerization unit EL 1 plus N₂.

LP/H and LP/HS transparent and pink-transparent

EL 1	2 min. 30 sec. to 3 min.
EL 1 plus N ₂	2 min. 30 sec. to 3 min. (without inert gas)
EL 2	3 min. to 3 min. 30 sec.
eLED.LP3	2 min.

LP/H beige

EL 1	2 min. 30 sec. to 3 min.
EL 1 plus N ₂	2 min. 30 sec. to 3 min. (without inert gas)
EL 2	3 min. to 3 min. 30 sec.
eLED.LP3	1 min.

LP/H flesh

EL 1	1 min. 10 sec. to 1 min. 20 sec.
EL 1 plus N ₂	1 min. 10 sec. to 1 min. 20 sec. (without inert gas)
EL 2	1 min. 20 sec. to 1 min. 30 sec.
eLED.LP3	1 min.

LP/H red and blue

EL 1	30 sec.
EL 1 plus N ₂	30 sec. (without inert gas)
EL 2	1 min. to 1 min. 10 sec.
eLED.LP3	15 sec.

ATTENTION: The mentioned polymerization times are guide values, they may vary depending on the size of the earmold and the transparency of the negative mold.

- Remove the black foil after polymerization.
- Pour the liquid material into a container that is impervious to light. **This material can be reused.**
- After having completely poured out the liquid material, put the negative mold and the blank but without the black foil into the UV-polymerization unit again. **We recommend placing ITE shells in the unit with the opening facing downward.** Inert gas is **not** required if you work with the UV-polymerization unit EL 1 plus N₂.

LP/H and LP/HS transparent and pink-transparent

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- Take the blank out of the negative mold and put it into a transparent container (REF 50107) filled with LP-Cleaner (REF 31300ff). Then put it into a UV-polymerization unit for the second polymerization. **If for the second polymerization inert gas is used, put the blank into a transparent box with cover (REF 51114).**
- EL 1 5 min. to 6 min.
EL 1 plus N₂ 5 min. to 6 min. with inert gas
EL 2 7 min. to 8 min.
eLED.LP3 5 min.
- Then remove the blank. If you worked with LP-Cleaner (REF 31300ff) rinse the blank under running water.

Treatment and lacquering

- The blank can now be shaped with the special egger carbide cutters.
- Before lacquering the surface, roughen it with abrasive paper, grit 180, (REF 40704) then clean and degrease it with Surface Cleaner (REF 31600ff). Afterwards you can coat the earmold/ITE shell under a suction unit with the LP/H lacquer (REF 30600ff), LP/H lacquer antibac (REF 30621ff) or LP/W lacquer (REF 30700ff).

Counter indications: Do not use if hypersensitive to acrylate/methacrylate.

Please follow the product information and the material safety data sheet.