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Indicatie

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Accessories for ear impressions

Indication for use

For sealing the auditory canal to protect the ear drum during impression-taking.

Contraindication

The product must not be used for patients suffering from a perforated eardrum, inflammation or damage to the ear. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e.g. allergies) or local disorders of sensation (e.g. irritation in the auditory canal) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Target Patient Group

Patients who need ear care.

Intended users

ENT specialist, hearing aid audiologist

Product description

Impression pads Standard/CIC

Foam cone with safety thread to protect the eardrum during impression-taking. The pads consist of flexible foamed material and seal the auditory canal by a safe fitting.

Vented Impression pads

The pads consist of flexible foamed material and seal the auditory canal by a safe fitting. Especially suitable for deep ear impressions. These pads are manufactured with a ventilation tube relieving air pressure while the impression material is injected into the ear as well as when the cured impression will be removed.

Batch number / Use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

Application

Before taking an impression the auditory canal and the eardrum have to be inspected. If any unusual features or abnormalities are found (e. g. irritation, perforation of the eardrum) an impression must not be taken. In cases of thick cerumen deposits and hairs in the auditory canal, the ear may require cleaning and the hairs trimmed. This may require referral to a suitable medical practitioner. In order to protect the eardrum, before the impression is taken an impression pad must be placed at the end of the external auditory canal in front of the eardrum. They have to be inserted with jaw tweezers and using the ear light. The impression pads CIC are suitable for children or narrow auditory canals.

Single-use advice

After making an impression, the impression pad with the hardened ear impression silicone cannot be used for making another impression.

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.

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Dreve Otoplastik GmbH is not liable for any damages caused by improper application of the product.

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Dreve Otoplastik GmbH no se hace responsable de los daños causados por una aplicación incorrecta del producto.