Lack L Lack L nano

Lacquer L Lacquer L nano

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Instructions for use Lacquer L

Impression lacquer for surface smoothing and sealing of silicone-based earmolds and impressions. Cold curing, fast drying.

Application

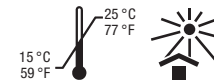
Prior to applying the lacquer (dipping or brush on) the impression must be absolutely clean, dry and free of grease! **Hint:** Use an appropriate surface cleaner (e.g. isopropyl alcohol). After shaking-off the excess of lacquer, let the impression (depending on coating thickness and humidity) dry in the air at room temperature 23 °C/73 °F for approx. 15 min. The surface of the impression shows now a glossy layer.

Caution

Avoid contact with eyes/skin. Wear protective gloves/eye protection. Do not breathe vapours.

Important

Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of Lacquer L, do not return residues of lacquer back into original bottle. Do not store in the refrigerator.

Safety advice

On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

PLEASE NOTE: During application of the lacquer use a suction system! For use by trained specialists. Please refer to the safety data sheet!

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CE 0483

Please follow the product information and
the material safety data sheet.

CE 0483

Sicherheitshinweise / Safety regulations:

Lack L / Lacquer L

DE: Enthält:

Methylcyclohexan, Xylol (o, m, p),
Triacetoxymethylsilan, Diocetylzinndi (acetat)

EN: Contains:

Methylcyclohexane, Xylene,
Triacetoxymethylsilane, Dioctyltin-di (acetate)

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EN: Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye damage. Harmful if inhaled. May cause respiratory irritation. May cause drowsiness or dizziness. May cause damage to organs through prolonged or repeated exposure. Toxic to aquatic life with long lasting effects.

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EN: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves / protective clothing / eye protection / face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER / doctor. In case of fire: Use Carbon dioxide (CO₂), Foam, Extinguishing powder to extinguish. Store in a well-ventilated place. Keep cool.

Sicherheitshinweise / Safety regulations:

Lack L nano / Lacquer L nano

DE: Enthält: Xylol (o,m,p),

Triacetoxymethylsilan, Diocetylzinndi (acetat)

EN: Contains: Xylene,

Triacetoxymethylsilane, dioctyltin-di (acetate)

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EN: Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye damage. Harmful if inhaled. May cause respiratory irritation. May cause damage to organs through prolonged or repeated exposure. Harmful to aquatic life with long lasting effects.

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EN: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear protective gloves / protective clothing / eye protection / face protection / hearing protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER / doctor. In case of fire: Use Carbon dioxide (CO₂), Foam, Extinguishing powder to extinguish. Store in a well-ventilated place. Keep cool.

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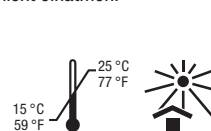
Instructions for use Lacquer L nano

Antibacterial sealing lacquer with silver particles.

Application

For all silicone earmolds. Prior to application (dipping/brushing) the earmold must be absolutely clean, dry and free of grease! **Hint:** Use an appropriate surface cleaner. After shaking-off the surplus of lacquer, let the earmold (depending on coating thickness and humidity) dry at 80 °C / 176 °F for approx. 20 min. or at room temperature 23 °C / 73 °F over night. The surface of the earmold shows now a glossy layer. Do not insert earmold until the lacquer is completely odorless.

Caution

Avoid contact with eyes / skin. Wear protective gloves / eye protection. Do not breathe vapors.

Important

Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of Lacquer L nano, do not return residues of lacquer back into original bottle. Do not store in the refrigerator.

Safety advice

On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

PLEASE NOTE: During application of lacquer use a suction device! For use by trained specialists. Please refer to the safety data sheet!

Caution

Federal U.S. law restricts this device to sale by or on the order of a licensed healthcare practitioner (or trained specialist personnel).