Bedienungsanleitung
Instructions for use
Mode d'emploi
Brugsanvisning

Injector control A

Elektromotorisches Handgerät für Doppelkartuschen
Electro-motive hand unit for double cartridges
Appareil manuel électrique pour cartouches doubles
Ledningsfri aftrykspistol til dobbeltpatroner

deutsch

english

français

dansk

Inhalt DE

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Contents EN

| | |
|--------------------------------------|----|
| Meaning of signs | 5 |
| Spare part list | 14 |
| Technical Data | 14 |
| Declaration of conformity | 15 |
| Safety advice | 16 |
| Description of the unit | 17 |
| Putting into service | 17 |
| Operation | 18 |
| Exchange of cartridges | 19 |
| Usage for S50 cartridges | 19 |
| Usage for S48 cartridges | 19 |
| Care and maintenance | 20 |
| General notes | 20 |
| Placing the product out of operation | 20 |
| Trouble shooting | 21 |
| Wiring diagram | 38 |

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Warning! Here you will find especially important notes and instructions whose disregard may provoke dangers of injury and accidents.

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Attention! Here you will find important information that you should observe.

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Spare part list

| | REF |
|--------------------------------------|----------|
| 1 Circuit board (electronic control) | 52438 |
| 2 Housing of the piston guide 2 pcs. | 52441 |
| 3 Tape drive unit | 52443 |
| 4 Molded recess 2 pcs. | 52439ROT |
| 5 Charger 220–230 V / 50–60 Hz | 1449LD |
| 6 Charger 115 V / 60 Hz | 1449LUSA |
| 7 NiMH Accu | 52433 |

Technical Data

| | |
|-----------------------|--|
| H x W x D | 200 x 220 x 65 mm |
| Weight | approx. 620 g with integrated accu |
| Voltage | 220–230V / 50–60Hz, 115V / 60Hz (Voltage / frequency see at type plate) |
| DC motor | 4Watt / 12V |
| min. multistage speed | 0.35 r / min – idle speed |
| max. multistage speed | 3.50 r / min – idle speed |

Declaration of conformity

According to EMV Directive (2014 / 30 / EU)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental safety and machine guidelines as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

| | |
|-------------------------------------|--|
| Denomination of the machine | Injector control A |
| REF | 15151, 15151A |
| Type of machine | electromotive handheld device for double cartridges |
| Applicable EC directives | 2014 / 30 / EU EMV-Directive |
| Applied harmonised standards | 2014 / 30 / EU EMV-Directive |
| EN 61000-3-2:2014 | Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current <= 16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014. |
| EN 61000-3-3:2013 | Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current <= 16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013. |
| EN 60601-1-2:2007 Ber1:2010-05 | Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic disturbances – Requirements and tests (IEC 62A/746/CD:2011) Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests (IEC 60601-1-2:2007, modified); German version EN 60601-1-2:2007, Corrigendum to DIN EN 60601-1-2 (VDE 0750-1-2):2007-12; German version CENELEC-Cor. :2010 to EN 60601-1-2:2007. |
| Transient emission | |
| EN 60601-1-2:2007 Ber1:2010-05 | “General requirements for basic safety and essential performance” Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters. |
| Immunity | |
| EN 60601-1-2:2007 Ber1:2010-05 | “General requirements for basic safety and essential performance” Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters. |

Safety advice

Attention! Read the following notes before connecting and putting the unit into service. The operating service and the function of the unit can only be guaranteed for, if the general safety and accident prevention instructions of the legislation as well as the safety instructions of the working instruction are observed.

1. This unit must only be used according to the present working instructions. We cannot be held liable for damages which have been caused by improper treatment resp. faulty manners of operation.
2. The indicated voltage on the type plate of the generator has to meet the voltage of the power source.
3. Never push the operating lever of the Injector control A to the front, i. e. in the direction of the double cartridge, when the unit is closed.
4. Never touch the generator and Injector control A with wet hands.
5. Plates and labels have to be maintained in a well legible condition and must not be removed.
6. Only those appliances and spare parts may be used which are released by the manufacturer. We are not responsible for damages caused by the use of other parts.
7. Do not lock the operation button of the unit.
8. The unit has to be examined before operation in view of proper condition. If the unit is not in proper condition, it must not be used and has to be marked accordingly.
9. Before cleaning and maintenance of the unit or the exchange of parts the plug has to be disconnected.
10. When cleaning the unit observe that no cleaning liquid gets inside the unit.
11. Do not intrude any objects into the unit.
12. The opening of the unit and repairs are only to be carried out by accordingly trained experts.
13. Keep the unit away from flammable liquids, solvents and strong heat sources, as these can damage the plastic housing.

Description of the unit

The Injector control A is an electromotive hand unit for the emptying of all current silicone-filled 48 and 50 ml double cartridges. For independent use the Injector control A is equipped in handle with an metal hydride accu. The driving speed of the unit

can be controlled in multi-speed by increasing or decreasing pressure exerted on the button. The integrated maximum power limit avoids the bursting of a cartridge when producing accidentally against a cured mixing canula.

Putting into service


Before operation ensure that the indicated voltage on the type plate meets the voltage of the generator.

In the case of factory-new units or when operating units after longer storage time, the accu of the Injector control A has to be charged first. For this put the plug of the cable into the socket of the Injector control A and then create a connection with the generator. The voltage control lamp in the power unit signalises whether the unit is ready to operate and shows the charging operation.

The charging time for a new resp. completely empty accu amounts to approx. 2.5 hours. With the charged accu approx. 10 double cartridges can be injected at the max.

In order to obtain a long life-time of the accu, the following basic points have to be observed:

- The charge contacts must not get in touch with metal or greased parts.
- A drain of the accu, as e. g. by short-cutting of the contacts, has to be avoided.
- Do only execute a new charging of the accu, if the performance of the Injector control A decreases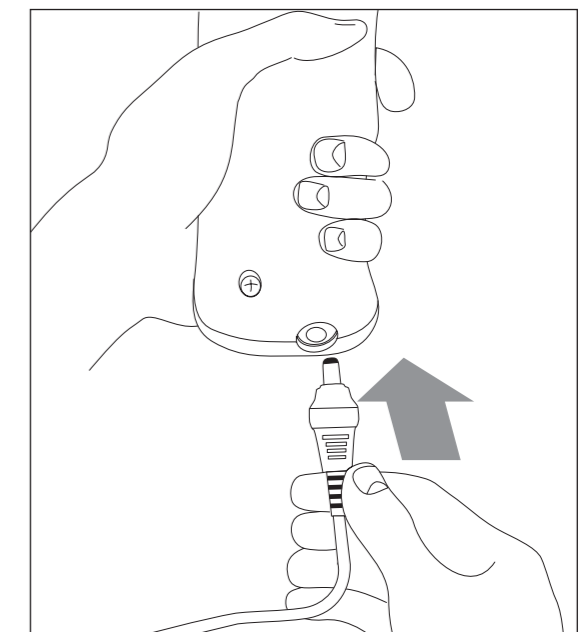


remarkably. When often charging an accu which has only been discharged partly, the operation time can be shortened considerably. However, an accu is a wear and tear part, its capacity and the related operating hours decrease after a certain time.


- Injector control A must not be loaded while conveying.

Operation


Open the upper part of the unit by pushing the two push-buttons at the same time and swiveling it upwards. The opening angle is restricted by the arrester.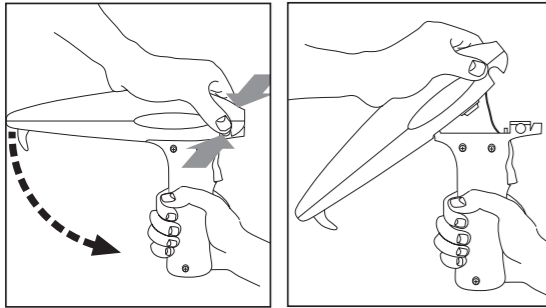

Pull the piston guide with the operating lever in the rear stud.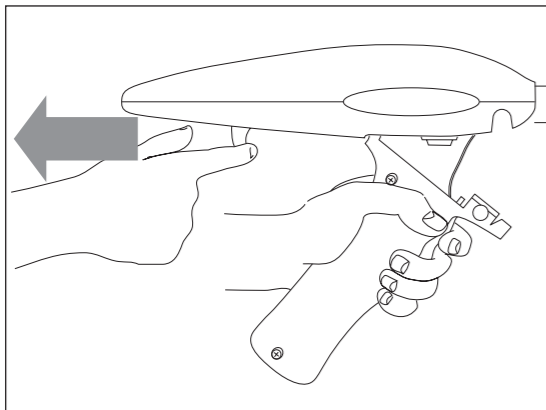

Insert the double cartridge into the guide rail and press to the stud.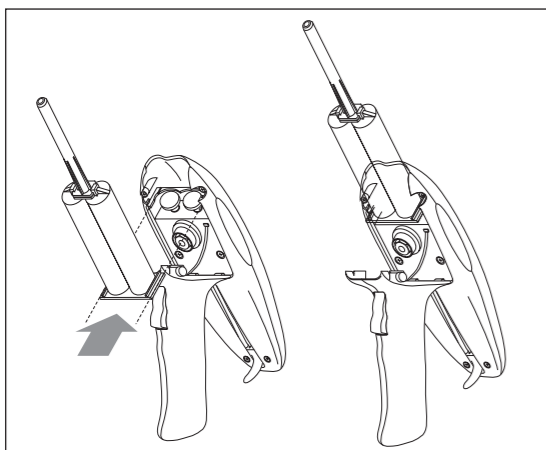

Afterwards, when the unit is opened, push the operating lever under slight pressure in direction of the double cartridge to the contact.

Attention! The operating lever must not be pushed to the front when the unit is closed.

Swivel the upper part of the unit downwards and let it snap hearably. Remove the protection cap of the double cartridge by turning it to the right by 90° and do not use again.

Press the mixing canula into the reception of the double cartridge and lock it by turning it to the right by 90°. The Injector control A is now ready to operate. The driving speed of the unit can be controlled in multispeed by increasing or decreasing pressure exerted on the button. After each pressing of the button the gear motor runs backwards for a short time in order to enable the silicone to relax.

Attention! Avoid touching the button unnecessarily or severally, as the provoked running backward may probably cause the tensioning band to break!

After having finished the filling process do not remove the mixing canula. Do only remove the mixing canula before the next use by means of a 90° turn to the right.

Attention! Never use a mixing canula with cured material!

If the injection of the impression material takes too much time the accu will have to be recharged.


Exchange of cartridges

Open the upper part of the unit by pushing the two push-buttons at the same time and swiveling it upwards. The opening angle is restricted by the arrester. Pull the piston guide with the operating lever in the rear stud. Take the empty double cartridge out of the guide rail. Go on as indicated in chapter operation.

Attention! The operating lever must not be pushed to the front, i.e. in direction of the double cartridge when the unit is closed.

Usage of S50 cartridges

When using the S50 ml-cartridge it has to be observed that the notch in the slide profile of the cartridge is placed towards of the upper part of the unit.

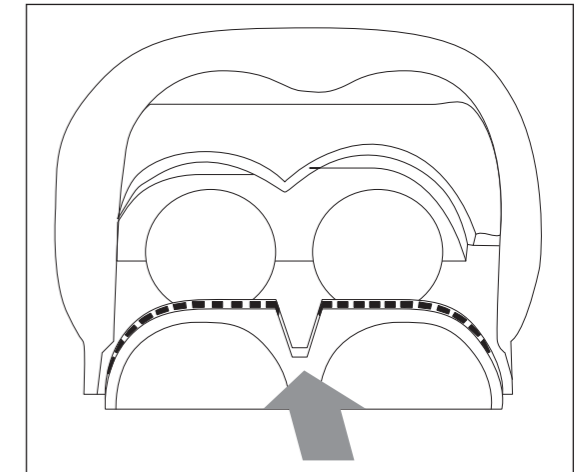Usage of S48 cartridges

If only the S48 ml cartridge system is used, the enclosed insert can be attached and fixed with superglue for an optimal stabilisation of the double cartridge (please see figure).

Care and Maintenance

Disconnect the unit from mains supply before executing care and maintenance!

Cleaning is effected with a mild cleaning liquid. The cleaning liquid must not get into the unit.

General notes

Repairs on the Injector control A may only be executed by experts. Disconnect the unit from mains supply at any rate.

Dreve company is only responsible for the safety, reliability and performance of the unit, if:

- the unit is exclusively used for the corresponding field of application.
- operation is only carried out by accordingly trained personnel.

- the unit is operated in accordance with these instructions for use.
- extensions, new adjustments, changes or repairs are executed by persons authorised by the company.

As we are currently developing our products, we reserve the right for technical changes.

Placing the product out of operation

The DIRECTIVE 2012/19/EU OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 4 July 2012 on waste electrical and electronic equipment (WEEE) lays down measures to protect the environment and human health. Waste electrical and electronic equipment marked as such may not be disposed of as unsorted municipal waste within the European Union. For detailed information regarding the professional disposal of WEEE please contact Dreve or your distributor.

Trouble shooting

| Problem | Cause | Aid |
|---|--|--|
| Device shows no function | Accu not charged | Connect Injector control A with the mains adapter of the charger and charge accu for 2.5 h. |
| Device shows no function, accu loaded | Accu defect | Send the device to the manufacturer |
| Mains adapter of the charger is in the socket, accu does not load | Accu defect | Exchange accu, send the device to the manufacturer |
| | Mains adapter of the charger defect | Exchange mains adapter |
| | Cable connection of mains adapter and Injector control A is defect | Check cable connection |
| | Socket without voltage | Check power socket |
| Unit produces the material slowly | Accu has been emptied | Charge accu |
| Motor runs and then stops | Double cartridge has been emptied | Insert new double cartridge |
| Motor runs and then stops | Cured material in old mixing canula | Apply new mixing canula |
| Unit does not produce, motor runs hearably | Upper part of the unit has not snapped into final position, coupling does not grip | Let upper part snap in hearably |
| Unit does not produce, motor runs hearably | Tensioning band is torn | The upper part of the unit has to be demounted by an expert, the tensioning band has to be exchanged |
| Emptied double cartridge cannot be taken out of the unit | The piston guide is not positioned at the rear stud | Pull piston guide at the operating lever up to the rear stud |

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According to EMV Directive (2014 / 30 / EU)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental safety and machine guidelines as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

Denomination of the machine Injector control A

REF 15151, 15151A

Type of machine electromotive handheld device for double cartridges

Applicable EC directives
2014 / 30 / EU EMV-Directive

Applied harmonised standards
2014 / 30 / EU EMV-Directive
EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current <= 16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.

EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current <= 16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.

EN 60601-1-2:2007 Ber1:2010-05 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic disturbances – Requirements and tests (IEC 62A/746/CD:2011) Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests (IEC 60601-1-2:2007, modified); German version EN 60601-1-2:2007, Corrigendum to DIN EN 60601-1-2 (VDE 0750-1-2):2007-12; German version CENELEC-Cor.: 2010 to EN 60601-1-2:2007.

Transient emission
EN 60601-1-2:2007 Ber1:2010-05 "General requirements for basic safety and essential performance" Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters.

Immunity
EN 60601-1-2:2007 Ber1:2010-05 "General requirements for basic safety and essential performance" Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters.

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Applied harmonised standards
2014 / 30 / EU EMV-Directive
EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current <= 16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.
EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current <= 16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.
EN 60601-1-2:2007 Ber1:2010-05 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic disturbances – Requirements and tests (IEC 62A / 746 / CD:2011) Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests (IEC 60601-1-2:2007, modified); German version EN 60601-1-2:2007, Corrigendum to DIN EN 60601-1-2 (VDE 0750-1-2):2007-12; German version CENELEC-Cor.: 2010 to EN 60601-1-2:2007.

Transient emission
EN 60601-1-2:2007 Ber1:2010-05 “General requirements for basic safety and essential performance” Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters.

Immunity
EN 60601-1-2:2007 Ber1:2010-05 “General requirements for basic safety and essential performance” Product standard for non life-sustaining devices, not operating, measuring or monitoring of physical parameters.

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