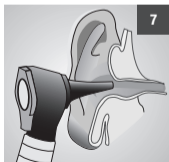
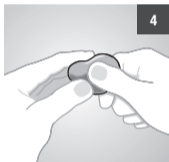
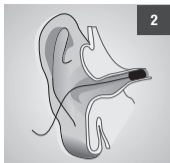
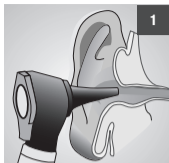
egger C/II

Ohrabformsilikone
Ear impression silicones

egger

DE	Gebrauchsanweisung	6	PT	Manual de instruções	32
EN	Instructions for use.....	8	RO	Instrucțiuni de utilizare.....	34
FR	Mode d'emploi.....	10	SV	Bruksanvisning	36
ES	Modo de empleo	12	SK	Návod na použitie.....	38
IT	Istruzioni per l'uso.....	14	SL	Navodila za uporabo.....	40
DA	Brugsanvisning	16	CS	Návod k použití	42
EL	Οδηγίες χρήσης.....	18	HU	Használati útmutató.....	44
HR	Upute za uporabu.....	20	RU	Инструкция по применению	46
LV	Lietošanas instrukcija.....	22	TR	Kullanım kılavuzu	48
LT	Naudojimo instrukcija.....	24	FI	Käyttöohje	50
NL	Gebruiksaanwijzing	26	ET	Kasutusjuhend	52
NO	Bruksanvisning	28	BG	Инструкция за употреба	54
PL	Instrukcja stosowania.....	30			

Technical Specifications

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Ear impression taking

Patient Target Group

Persons for whom an ear impression is to be created. Catalyst component contains alkyl stannosiloxanes: Do not use for the treatment of pregnant/breastfeeding women and children.

Intended Users

ENT specialist / hearing aid acoustician

Technical data

See page 5

1. Ear preparation

Before taking the impression, carefully examine the auditory canal and the eardrum (Fig. 1). In case of any pathological findings (e.g. injuries, inflammation or perforation of the eardrum, etc.), a specialist must be consulted before an impression may be taken. The auditory canal has to be cleaned and depilated. In case of insufficient cleaning, remaining cerumen can cause retarded setting resp. inhibition of setting process.

Before taking the impression, the eardrum must always be protected with an impression plug placed at the end of the external auditory canal in front of the eardrum (Fig. 2). Now egger C/II is injected directly into the prepared ear of the patient as described under point 2.

2. Mixing and dispensing

Put putty onto the hand and flatten out (wear protective gloves). Press rim of measuring scoop into putty. Apply catalyst paste along impression of measuring rim up to the marking (= 6 cm) (Fig. 3). Mix together until a uniform colouring is achieved (Fig. 4).

Fill the homogeneous mixture into an impression syringe and apply it slowly and bubble-free into the ear canal until the concha is filled completely (Fig. 5). Keep the end of the cannula immersed on the surface of the material during dispensing. After curing remove the impression carefully from the ear (Fig. 6). Re-examine the ear (Fig. 7).

See figures on page 2

Safety Information

- Only for the specified intended use by trained specialists.
- Do not bring into contact with addition-curing silicones.
- Cured impression materials are chemically resistant – avoid stains on clothing.
- Do not leave any residues of the impression material in the ear.
- Careful preparation (item 1) and cleaning of the ear canal are necessary before taking any impressions.
- After each impression taking, careful examination of the auditory canal is necessary.
- The processing instructions and precautionary measures must be observed. Otherwise, irreparable damage to the hearing organ or eardrum may result.
- Wear suitable personal protective equipment during the finishing of the cured material.
- Refer to the relevant safety data sheet for hazard and safety information.

Notes

- DETAX shall not be held liable for any damage caused by misuse.
- Read and understand safety data sheet!

For users and/or patients:

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

Contraindication

Contains alkyl stannosiloxanes.

Do not use this material in case of known allergies to any of the ingredients, or contact allergies. Silicone based materials are proven a million times. On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik + Labortechnik GmbH
DETAX GmbH