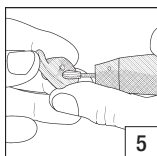
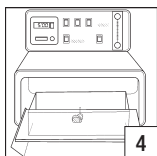
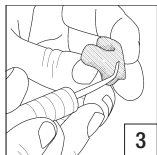
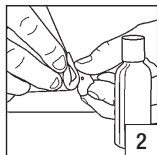
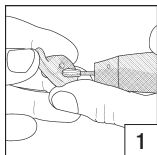
egger LP/H
egger LP/H Xpress





Modellierpasten zur Reparatur
von harten Otoplastiken
Modeling pastes for the repair
of hard earmoulds

egger

Modifications/ Repairs

Ventings in ITE shells

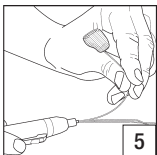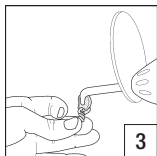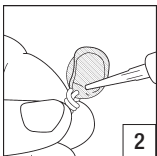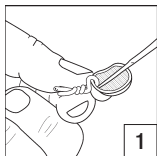DE	Gebrauchsanweisung	4	PT	Manual de instruções	43
EN	Instructions for use.....	7	RO	Instrucțiuni de utilizare.....	46
FR	Mode d'emploi.....	10	SV	Bruksanvisning	49
ES	Modo de empleo	13	SK	Návod na použitie.....	52
IT	Istruzioni per l'uso.....	16	SL	Navodila za uporabo.....	55
DA	Brugsanvisning	19	CS	Návod k použití	58
EL	Οδηγίες χρήσης.....	22	HU	Használati útmutató.....	61
HR	Upute za uporabu.....	25	RU	Инструкция по применению	64
LV	Lietošanas instrukcija	28	TR	Kullanım kılavuzu	67
LT	Naudojimo instrukcija	31	FI	Käyttöohje	70
NL	Gebruiksaanwijzing	34	ET	Kasutusjuhend	73
NO	Bruksanvisning	37	BG	Инструкция за употреба	76
PL	Instrukcja stosowania	40			

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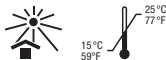
Lagerung:

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Intended Use

Modeling pastes for the repair of hard earmoulds

Indication

Applicable for hard earmoulds based on powder monomer (2-component (meth)acrylates) or 1-component UV-light-curing material.
Production of the venting channels for hard earmould shells.

Patient Target Group

Persons for whom an earmould is to be created.

Intended Users

Hearing aid acoustician, earmould laboratory

General product information

The egger LP/H modeling pastes can be used in a variety of ways:

- Repair of BTE and ITE earmoulds made of light-curing polymers or other hard acrylics
- Reduction or closing of venting channels
- Subsequent attachment of supports
- Application of a color marking
- Preparation and modification of ventings in ITE earmoulds

With the Xpress modeling paste, you can create ventings or carry out repairs or modifications with a time saving of up to 50 % at consistently high quality. The Xpress modeling paste has been specially adapted to the wavelength range of the Megalux CS light gun (egger order no 51550) in order to achieve an optimum polymerization time. The handy tubes or cartridges ensure efficient and clean application.

Processing instructions

Processing is possible in the following egger EL light units:

egger EL 1	egger order no 51100
egger EL 1 plus N2	egger order no 51110
egger EL 2	egger order no 51200
egger EL 3	
egger eLED.LP3	egger order no 51350
Light gun	egger order no 51550

Please pay attention to the device description applicable to the device you are using, and observe the polymerization times specified below.

Polymerization times

egger EL light units:	Polymerization time:
EL 1, EL 1 plus N2, EL 2, EL 3, eLED.LP3	2 – 6 minutes
Light gun	20 – 180 seconds

Modifications / repairs (see page 2)

1. The area on the earmould to be treated must in each case be first roughened with corundum paper, grit 180 (egger order no 40704).
2. Then clean and degrease carefully with the surface cleaner (egger order no 31600).
3. Apply the modeling paste directly from the tube or with the help of a modeling tool (egger order no 31215), and smooth the transitions to the substrate.
4. Then, depending on the applied layer thickness, cure for approx. 2 – 6 minutes in the light unit or for 20 – 180 seconds with the light gun.
5. After polymerization, thoroughly wipe the surface of the earmould with a cloth soaked in surface cleaner (egger order no 31600).
6. If necessary, finish the earmould with a milling cutter and prepare it for varnishing.

Ventings in ITE shells (see page 2)

1. Roughen the inner surface of the ITE shell with a suitable rose drill bit (egger order no 70100). Then insert the vent hose with wire (egger order no 92100) into the ITE shell and twist them together on the outside. Use the modeling tool (egger order no 31215) to press the vent hose with wire onto the inner wall of the shell.
2. Cover the vent hose (egger order no 92100) with modeling paste beige (egger order no 31000) or modeling paste Xpress (egger order no 31020).
3. Then cure for approx. 20 – 180 seconds with the light gun (egger order no 51550), depending on the thickness of the layer applied. After curing, carefully pull out the vent hose with wire (egger order no 92100).
4. Then post-polymerize the ITE shell with venting applied in the LP-Cleaner bath or under gas (for EL1 plus N2) for 4 – 6 minutes in the light unit.
5. After this, glue the faceplate with Uniglu (egger order no 39503) and, using the vent light rod (egger order no 26611), mark the venting outlet and open it with a drill bit of appropriate size.

Safety Information

- Only for the specified intended use by trained specialists.
- Avoid direct skin contact with the liquid material!
- During processing, hands must be protected by wearing suitable gloves.
- Refer to the relevant safety data sheet for hazard and safety information.

Notes

- DETAX shall not be held liable for any damage caused by misuse.
- Re-seal the tube immediately after use.
- Read and understand the safety data sheet!

For users and/or patients:

Any serious incident that has occurred in connection with this product must be reported immediately to incident@detax.com and to the competent public authority of the country in which the user and/or patient resides.

Contraindication

Contains (meth)acrylates and phosphine oxide.

Ingredients of the modeling pastes egger LP/H and egger LP/H Xpress may cause allergic reactions in accordingly predisposed persons. In such a case, do not use the product any further.

Side Effects

Product may cause allergic reactions.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

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egger Otoplastik+ Labortechnik GmbH
DETAX GmbH