

BEST PRACTICE

PERFECTDOME CHECKLIST ✓



Use a cleaning wipe for your hearing aids to remove external dirt before placing them in the PerfectDome.



When using rechargeable hearing aids, connect the charging station to the PerfectDome. This ensures the hearing aids are charged while being maintained.



Make sure the lid is properly closed before turning on the PerfectDome. This is crucial for the optimal functioning of the drying and UV-C features.



Let the PerfectDome complete its full cycle for the best results. The device will automatically shut off once the cycle is finished.



Unplug the PerfectDome when it's not in use, especially if you want to save energy. This is also better for the environment.



Clean the PerfectDome weekly and ensure the interior stays dry for a long lifespan and optimal hygiene.



Clean and dry your hearing aids every evening with the PerfectDome. This enhances hygiene, improves comfort, and prevents malfunctions.
